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Instructions for use
Flexible hypodermic needle and Injectra

IFU Nr.33-A Injekt EN 03/23.02.2016

Please note: These medical products may be obtained and used only by trained medical personnel.

- Description**
Hypodermic needle for injecting substances in the bladder region.
- Contents and packaging**
- Flexible hypodermic needle with or without Tuohy Borst adapter, with marking from the tube and Luer connector for needle alignment (grinding)
- Injectra hypodermic needle with retractable needle tip
- Disposable packaging with instructions for use
- Intended use**
Injecting substances into the urogenital tract.
- Indication**
- Vesicoureteral reflux (VUR)
- Overactive bladder with or without incontinence (OAB)
- Contraindications**
- Severe urinary tract infection
- Increased tendency to bleed or coagulation disorders
- Inflammatory disorders
- Possible side effects**
Hemorrhages, vascular lesions, hematuria, infection (ascending), pain around the puncture site, bacteremia.
- Instructions**
When using the product, the general hygiene rules for injections apply.
Flexible hypodermic needle
1. Preparing the disposable syringe with the substance to be administered.
2. Removing the protective tube of the flexible hypodermic needle.
3. Inserting the flexible hypodermic needle into the cystoscope.
4. Removing the front, white protective hose.
5. Connecting the disposable syringe and injecting the fluid until the needle is free of air bubbles.
6. Draw the flexible hypodermic needle back into the cystoscope until the cannula no longer protrudes out of the cystoscope.
7. Inserting the cystoscope up to the bladder. Make sure the tip of the needle is not protruding from the cystoscope.
8. Puncturing and injecting the substance at the desired application site(s).
Attention: there should not be any blood vessels at the injection site. The lowest possible injection angle must be taken in order to avoid piercing the bladder wall.
9. After application, completely withdraw the flexible hypodermic needle from the cystoscope's working channel.

- Injectra hypodermic needle**
1. Preparing the disposable syringe with the substance to be administered. Make sure that the safety clip on the Injectra is properly attached to the handle to ensure the needle is fully retracted.
2. Push the Injectra hypodermic needle through the endoscope's working channel until the tube protrudes approx. 1 cm from the working channel.
3. In order for the needle to be easily pushed out of the tube, the endoscope or Albaran lever is angled and the tube placed on the intended injection site.
4. The Injectra hypodermic needle can now be pulled out or injected.
5. Place the disposable syringe on the Luer lock of the Injectra hypodermic needle.
6. To move the needle out of the external catheter at the desired position, first remove the safety clip and then push the syringe plunger forward until the locking mechanism locks it into the handle.
7. After administering the medication, press the unlock button (white dot on the handle of the hypodermic needle) which retracts the needle into the external catheter.
8. Now pull the Injectra hypodermic needle out of the endoscope's working channel.

To ensure the Injectra hypodermic needle is completely retracted into the tube, reattach the safety clip to the handle.

- Possible complications and/or risks**
- Puncturing blood vessels in the wall of the bladder can lead to hematuria.
- Incorrect puncture
- Complications due to the application of cystoscopic examinations.
- Warnings / Precautions**
- This product is only sterile if the packaging is undamaged and unopened.
- For single use only!
- Do not use needles after the date of expiry has passed.
- Do not wipe the product and set components with alcohol/disinfectant agents. This may damage the surface.
- All components must be carefully checked for compatibility and integrity before use.
- The Tuohy Borst adapter should not be screwed too tight in order to prevent damage to the tube.
- The flexible hypodermic needle may only be used with a rigid cystoscope in order to prevent the needle from damaging the working channel.
- The tube of the flexible hypodermic needle may kink due to the pressure being too high while puncturing.
- The tip of the Injectra needle must not be pushed out of the tube into the working channel as this may lead to damage to the tip of the needle and/or the working channel.
- Hypodermic needles are intended for a temporary service life of ≤ 60 minutes.
- Interaction with other drugs**
The flexible hypodermic needle could potentially discolor when applying urinary examination methods that use dye.
- Transport and storage conditions**
The products may be transported and stored only in the packaging intended for this. There are no further specific requirements for transport. Products must be stored dry and protected from direct sunlight, in the temperature range of 5 - 30 degrees Celsius.
- Disposal**
After use, this product may pose a biological hazard. Handling and disposal must be carried out in accordance with recognized medical procedures and be completed pursuant to applicable legal regulations and guidelines.

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