	DE	GB	FR	IT	ES	PT	GR	AE	CN	JP	CZ
	Hersteller	Manufacturer	Fabricant	Fabbricante	Fabricante	Fabricante	Κατασκευαστής	الجهة المنتجة	製造商	メーカー	Výrobce
	Herstellungsdatum	Date of manufacture	Date de fabrication	Data di fabbricazione	Fecha de fabricación	Data de fabricaço	Ημερομηνία παραγωγής	تاريخ التصنيع	生产日期	製造日	Datum výroby
	EU-Konformitätszeichen (Nummer gibt die Registrierungsnummer der benannten Stelle an)	EU conformity symbol (number indicates the registration number of the notified body)	Marquage CE de conformité (les chiffres indiquent le numéro d'identification de l'organisme notifié)	Marchio di conformità UE (il numero indica il numero di registrazione dell'organismo notificato)	Marca de conformidad de la UE (el número indica el número de registro del organismo designado)	Símbolo de conformidade com a legislação europeia (indica o número de registo do organismo notificado)	Σύμβολο συμμόρφωσης ΕΕ (ο αριθμός είναι ο αριθμός του αριθμού του καταχωρημένου φορέα)	رمز مطابقة الاتحاد الأوروبي (تحتوي الرقم إلى رقم تسجيل الهيكل المعنى)	欧盟合格标志 (数字指所相关认证机构的注册号)	CEマーキング (番号は通知機関の登録番号を示す)	EU značka shody (číslo je číslo registračným ústnem jmenovaného místa)
	Bestellnummer	Order number	Numéro de commande	Numero d'ordinazione	Nº. de pedido	Número do pedido	Αριθμός παραγγελίας	رقم طلب الشراء	订单号	注文番号	Objednací číslo
	Chargenbezeichnung	Batch code	Numéro de lot	Numero del lotto	Código de lote	Número de lote	Χαρακτηριστικός παρτίδας	اسم الطليقة	批号	バッチコード	Označení šarže
	Verwendbar bis	Use by	Utilisable jusqu'au	Utilizzabile entro	Fecha de caducidad	Prazo de validade	Ημερομηνία λήξης	مصلحة للاستخدام حتى	有效期至	有效日期	Použitelné do
	Sterilisation mit Ethylenoxid	Sterilisation with ethylene oxide	Sterilization par oxyde d'éthylène	Sterilizzazione con ossido di etilene	Esterilización con óxido de etileno	Esterilização por óxido de etileno	Αποστείρωση με αιθυλοξείδιο	تعقيم من خلال أكسيد الإيثيلين	环氧氣乙烷灭菌	エチレンオキシド滅菌	Sterilizace ethylenoxidem
	Nicht erneut sterilisieren	Do not sterilize	Ne pas restériliser	Non sterilizzare nuovamente	No volver a esterilizar	Não esterilizar	Δεν επαναποστείρωται	لا تعقم مرة أخرى	不可重复灭菌	滅菌済しない	Nesterilizujte opakovaně
	Nicht zur Wiederverwendung	Do not reuse	Ne pas réutiliser	Non riutilizzabile	No reutilizar	Não reutilizar	Δεν επαναχρησιμοποιείται	غير مسموح بإعادة استخدامها	一次性使用	再使用禁止	Není určeno k opakovanému použití
	Vorsicht! Bitte Warnhinweise beachten	Caution! Please observe warnings!	Attention ! Respecter les mises en garde	Attenzione! Si prega di rispettare le avvertenze	Precaución! Tener en cuenta las indicaciones	Cuidado! Tener os avisos!	Προσοχή! Τένει ο οδηγό χρήσης	تنبيه! يرجى قراءة التعليمات التحذيرية	注意！請遵守所有警告說明	注意！全ての警告をお守りください	Pozor! Dodržte prosím výstražné pokyny
	Gebrauchsanweisung ist zu beachten!	Operating instructions must be observed!	Respecter le mode d'emploi !	Rispettare le istruzioni per l'uso!	Tener en cuenta las instrucciones de uso!	As instruções de operação devem ser respeitadas!	Πρέπει να τηρούνται οι οδηγίες χρήσης!	يجب مراعاة الإرشادات الاستخدام!	務必遵守使用說明！	使用説明書の内容をお守りください！	Dodržte návod k použití!
	Produkt nicht bei Personen anwenden die einen Herzschrittmacher oder implantierten Defibrillator tragen!	Do not use this product on patients with a pacemaker or implanted defibrillator	Ne pas utiliser le produit sur des personnes portant un stimulateur cardiaque ou un défibrillateur implanté !	Non utilizzare il prodotto per i pazienti portatori di pacemaker o defibrillatore impiantato!	¡El producto no debe ser utilizado por personas que lleven implantado un marcapasos o un desfibrilador!	Não utilize este produto em doentes com marca-passo ou desfibrilador implantável!	Μην χρησιμοποιείτε το προϊόν σε άτομα που φέρουν εμφυτευμένο ή εμφυτεύσιμο στήθασ!	لا تستخدم المنتج من قبل الأشخاص الذين يعانون من مشاكل القلب القسطري أو مزيل الرجحان أو القليل للزرع!	不要將產品用于配戴心脏起搏器或植入式電擊除颤器者身上！	心臓ペースメーカーまたは体外式心臓起動器を使用されている方の近くで本製品をご使用にならないでください。	Nepoužívejte v blízkosti osob, které nosí kardiostimulátor nebo implantovaný defibrilátor!
	Keine MRT-Untersuchung bei Trägern dieses Produktes!	No MRT examinations on patients using this product!	Ne pas soumettre les personnes portant ce produit à un examen RM !	Non sottoporre i pazienti portatori di questo prodotto ad un esame TRM!	¡No realizar ningún análisis con tomografía por resonancia magnética (TRM) si consume este producto!	Proibido a tomografia de ressonância magnética em doentes usando este produto!	Απογορεύεται κάθε είδους μαγνητικής τομογραφίας, για άτομα που φέρουν αυτό το προϊόν!	لا يخضع نقلي الأشخاص الذين يحملون المنتج المغناطيسي!	不要將配戴該產品的病人送入MRI檢查！	本製品を使用されている方の近くでMRI検査を行わないでください！	Nesměle tohoto výrobku nesmí podstoupit vyšetření MRT!
	Bei beschädigter Verpackung nicht verwenden	Do not use if package is damaged	Ne pas utiliser si l'emballage est endommagé	Non usare se la confezione è danneggiata	No utilizar en caso de embalaje dañado	Não utilize se a embalagem estiver danificada	Να μην χρησιμοποιηθεί εάν η συσκευασία είναι ελαττωματική	لا تستخدم في حالة التلف	包裝有損時不可使用	包裝に破損がある場合、使用しないで下さい	Nepoužívejte, jestli obal poškozený
	Trocken aufbewahren	Keep dry	Conservar au sec	Conservare in luogo asciutto	Conservar en un lugar seco	Manter seco	Φυλάξτε σε στεγνό μέρος	يخفظ في مكان جاف	儲存于干燥处	乾燥した場所	Skladujte v suchu
	Oben	Up	Haut	Sopra	Amba	Em cima	Επάνω	أعلى	向上	上方	Nahoru
	Temperaturbegrenzung	Temperature limit	Limite de température	Limite di temperatura	Limite de temperatura	Limite de temperatura	Όριο θερμοκρασίας	تحديد درجة الحرارة	溫度限制	溫度制限	Omezení teploty
	Von Sonnenlicht fernhalten	Keep away from sunlight	Tenir éloigné de la lumière du soleil	Tenere lontano dai raggi del sole	Mantener apartado de la luz solar	Proteger de luz solar	Προστατέψτε από την ηλιακή ακτινοβολία	يخفظ بعيداً عن ضوء الشمس	避免阳光直射	日光から遠ざけてください	Udržujte z dosahu slunečního svitu
	Zulässige Stapelhöhe	Stacking limit by mass	Charge de perbage autorisée	Canso di impilaggio consentito	Carga aptible permitida	Carga máxima sobre a embalagem	Επιτρεπτό φορτίο στοβάθος	يسمح بالتحميل عليه	允許最大堆疊重量	重量による積上げ制限	Povolené stohovací zatížení
	Verschreibungspflichtig (US-Symbol)	Available only on prescription (US symbol)	Prescription médicale obligatoire (symbole utilisé aux États-Unis)	Soggetto a prescrizione medica (simbolo US)	De prescripción obligatoria (símbolo US)	Disponível somente mediante receita médica (símbolo US)	Υποχρεωτικά συνταγογραφούμενο (σύμβολο ΗΠΑ)	بتوفر حسب الوصف (رمز الولايات المتحدة)	仅适用于处方治疗 (美国标识)	処方薬 (米商シボル)	Pouze na předpis (symbol USA)

## IFU Nr. 11-A

DE	GB
<b>Gebrauchsanweisung</b> <b>Nierenzystenpunktionssset</b> IFU Nr.11-A Fistofix DE 00/02.03.2016	<b>Instructions for Use</b> <b>Renal cyst puncture set</b> IFU Nr.11-A Fistofix EN 00/02.03.2016

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- 1. Beschreibung**  
Fistofix renal cyst puncture set
- 2. Contents and packaging**  
- Pigtail catheter with stretching sleeve  
- 2-part puncture needle  
- Double packaging  
- Instructions for use
- 3. Intended use**  
Percutaneous puncture of renal cysts and short-term drainage.
- 4. Indications**  
- Renal cysts
- 5. Contraindications**  
- Coagulation disorders  
- Severe inflammation  
- Dermatological illness in the area of insertion
- 6. Possible side effects**  
Hemorrhages, vascular lesions, parenchyma loss, hematuria, infection (ascending), pain around the puncture site, bacteremia.
- 7. Instructions**  
**Note:**  
The sharpened tip of the cannula may not damage or puncture the interior of the pigtail catheter.

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Please note: This medical product may be obtained and used only by trained medical personnel.

- Description**  
Fistofix renal cyst puncture set
- 2. Contents and packaging**  
- Pigtail catheter with stretching sleeve  
- 2-part puncture needle  
- Double packaging  
- Instructions for use
- 3. Intended use**  
Percutaneous puncture of renal cysts and short-term drainage.
- 4. Indications**  
- Renal cysts
- 5. Contraindications**  
- Coagulation disorders  
- Severe inflammation  
- Dermatological illness in the area of insertion
- 6. Possible side effects**  
Hemorrhages, vascular lesions, parenchyma loss, hematuria, infection (ascending), pain around the puncture site, bacteremia.
- 7. Instructions**  
**Note:**  
The sharpened tip of the cannula may not damage or puncture the interior of the pigtail catheter.

- Remove the internal mandrin of the cannula.
- Stretch the pigtail with the stretching sleeve located on the catheter.
- Insert the outer tube of the cannula into the catheter until it reaches the Luer-Lock connector.
- Put the cannula together, re-insert the inner part of the cannula.
- Remove the stretching sleeve through the tip of the catheter.
- The catheter is now ready for use.

- Technical execution of cyst puncture:**  
- Location of the kidney/cyst under ultrasound monitoring  
- Puncture and drainage of the cyst
- 8. Possible complications and/or risks**  
- Puncturing blood vessels in the wall of the kidney can lead to hematuria.  
- Incorrect puncture  
- Incrustation, blood clotting, and kinks in the catheter can lead to partial or total obstruction of the suprapubic catheter with the risk of infected hydronephrosis, and in extreme cases urosepsis.  
- Dislocation of the catheter.  
- Risk of perforation of the kidney and the renal pelvis.
- 9. Instructions for use / precautions**  
- This product is only sterile if the packaging is undamaged and unopened.  
- For single use only!  
- All components must be carefully checked for compatibility and integrity before use.  
- Do not use products after the date of expiry has passed.  
- Do not wipe the product and set components with alcohol/disinfectant agents. This may damage the surface.  
- Do not damage the catheter with sharp surgical instruments – this increases the risk of breakage.  
- The sharpened tip of the cannula may not be used to stretch the pigtail as this may damage or puncture the inside of the catheter.  
- The needle can be bent due to excessive force while puncturing. The needle can be stabilized by supporting or fixing it in the puncture area, thereby preventing any bending or kinking.  
- This product is intended for a temporary service life of ≤ 30 days.
- 10. Interaction with other drugs**  
The catheter could potentially discolor due to the concomitant administration of medication containing dye-based urine presentation media, or even as a result of contact with urine.
- 11. Transport and storage conditions**  
The products may be transported and stored only in the packaging intended for this. There are no further specific requirements for transport.  
Products must be stored dry and protected from direct sunlight, in the temperature range of 5 - 30 degrees Celsius.
- 12. Disposal**  
After use, this product may pose a biological hazard. Handling and disposal must be carried out in accordance with recognized medical procedures and be completed pursuant to applicable legal regulations and guidelines.

FR	IT
<b>Mode d'emploi</b> <b>Kit de ponction de kyste rénal</b> IFU Nr.11-A Fistofix FR 00/02.03.2016	<b>Istruzioni per l'uso</b> <b>Set puntura cisti renale</b> IFU Nr.11-A Fistofix IT 00/02.03.2016

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