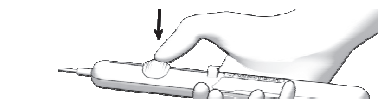
IFU Nr. 25-D

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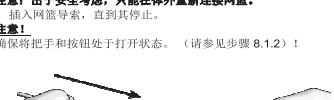
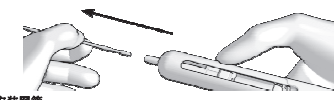
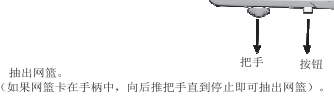
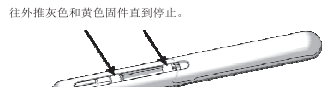
Please note: This medical product may be obtained and used only by trained medical personnel.

- 1. Description
2. Contents and packaging
3. Intended use
4. Indications
5. Contraindications
6. Possible side effects
7. Instructions

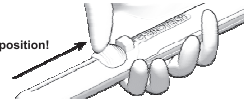
7.2.2. Push the slider down until it comes to a stop.

7.2.3. In the pressed state, pull the slider until it catches and hold in position.

Attention!

Hold the slider in this position!

7.2.4. At the same time, push down the gray push button.

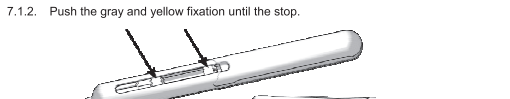
The stone basket is inserted into the target organ using the uretero-roscope. By withdrawing the tube covering, the basket is released at the tip of the instrument and unfolds.


In the event that the instrument gets stuck in the ureter, it is possible to release the handle from the basket and withdraw the endoscope, thereby unblocking the working channel.

7.1. Releasing the basket

- 7.1.1. Remove the protective cover
7.1.2. Push the gray and yellow fixation until the stop.

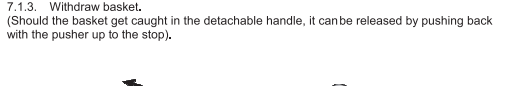7.1.3. Withdraw basket. (Should the basket get caught in the detachable handle, it can be released by pushing back with the pusher up to the stop).


- 7.2. Fixing the basket in place
7.2.1. Insert the basket wire until it comes to a stop.

Note: For safety, only reconnect the basket outside of the body. Ensure that the slider and push button are opened until the stop.

- 7.2.1. Insert the basket wire until it comes to a stop.

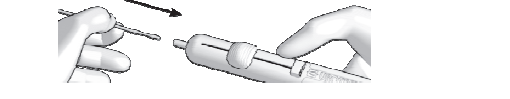Attention! Ensure that the slider and push button are opened until the stop.

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