DE

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EN

Steerable and non-steerable ureteral stent sets Instructions for use

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IFU Nr. 01-A

EN

Instructions for use Steerable and non-steerable ureteral stent sets

IFU Nr. 01-A Endo-J EN_03/03.04.2018

Please note: These medical products may only be obtained and used by trained medical personnel.

- 1. Description Double-J ureteral stents and stent sets for urinary diversion.
2. Contents and packaging Ureteral stent or tumor stent (with metal spiral or from a hard shaft with soft pigtail)...

Table with 3 columns: Section, Description, and Action. Contains instructions for insertion, positioning, and disconnection.

Removing a ureteral stent

- 1. Insert the cystoscope into the bladder
2. Retract the guide wire and pushing it forward again can cause the tip of the guide wire to exit through the holes in the ureteral stent...
3. Possible complications and/or risks Dislocation of the ureteral stent, Fragmentation of the product with subsequent surgical or endoscopic removal...

Table with 3 columns: Type 1, Type 2/Visiomatic, Dirigenta. Contains instructions for insertion, positioning, and disconnection.

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