DE

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Instructions for use
Transurethral OP catheter

GANL Nr. 40-A_EN 02/05.08.2019

Please note: This medical product may only be obtained and used by trained nursing and medical personnel.

1. Description

Two and three lumen transurethral OP catheter with balloon catheters made from medical grade silicone or latex with silicone elastomer coating, with various tips, lengths and balloon sizes.

2. Contents and packaging

- 2-way balloon catheter
- 3-way balloon catheter
- 3-way double balloon catheter
- Neobladder catheter
- Double packaging and instructions for use

3. Intended use

Temporary transurethral urine drainage, tamponade and possibility of bladder irrigation with 3-way balloon catheters.

4. Indication

Therapeutic Indications

- Intra-operative and post-operative urine drainage
- Bladder emptying disorders
- For transurethral bladder catheterization
- For tamponade and/or lavage following resections in the lower urinary tract

5. Contraindications

The balloon catheter may not be used in the case of known allergies to the respective catheter material (silicone or latex).

6. Side effects

- Pressure ulceration or inflammation of the urethral mucosa
- Catheter-induced infections (bacteriuria)
- Induction of squamous cell carcinoma of the bladder
- Catheter incarceration and obstruction (through sticky products from the inflamed bladder mucosa, such as urothelial cells and blood components, which adhere to the outer and inner catheter wall.)
- Urethral injury
- Hematuria

7. Instructions

When using this product, the general hygiene rules applicable to catheterization must be complied with.

Insertion technique (men)

The size of the balloon catheter is selected on an individual patient basis in accordance with medical standards.

Conventional medical lubricant is drizzled onto the urethral meatus using a disposable syringe. The external urethral opening is then widened manually, the cone positioned and the lubricant instilled by extending the head. To ensure that the lubricant does not leak out, the urethra should be held compressed for a moment or closed off with a penis clamp.

With the left hand, the penis is then held lateral to the glans penis. The catheter is then inserted with the sterile sheathing or sterile forceps. Its free end is grasped between the ring finger and thumb and secured in this way. On catheters with curved tips, the tip must point upwards. The catheter is then advanced smoothly past the sphincter with constant checks until it is inside the bladder and urine is draining out.

The catheter is then advanced a further 5 cm and, using the valve and the Luer attachment on a syringe, blocked through the second lumen and adjusted in the bladder by pulling it backwards.

Once the catheter has been inserted, it is connected via the funnel to a one-way connection on a urine bag and firmly pressed.

Removal of the catheter

- The balloon must be completely drained before removal. To do this, place an empty syringe on the valve and aspirate the fluid until a vacuum forms. The catheter is then carefully removed.

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