Stone basket
Instructions for use

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Instructions for use Stone basket

GANL Nr. 25-D_EN /02/20.06.2016

Please note: This medical product may be obtained and used only by trained medical personnel.

- Description**
 - Nitinol stone basket with scissors handle.
- Contents and packaging**
 - Nitinol stone basket with scissors handle: length 90 cm or 120 cm; basket with four or five wires, straight or helical; basket type: standard, tipless, flexible
 - Packaging, instructions for use
- Intended use**
 - Stone extraction from the ureter and renal pelvis or renal calices under ureterorenoscopic visual guidance.
- Indications**
 - Stone baskets catch and eliminate urinary stones with a rigid or flexible endoscope during a transurethral intervention for the following indications:
 - Removal of distal stones
 - Removal of higher lying stones after consideration of other methods (e.g. shock wave lithotripsy or laser disintegration) or if a "wait and see" attitude does not lead to success
- Contraindications**
 - Contraindications result from the restrictions for the uretero-renaloscopy.
- Possible side effects**
 - Injury of the kidney (bleeding)
 - Ureter injury during the procedure (perforation, bleeding)
 - Local fibrosis from extravasation of urine
 - Formation of a ureteral stricture
 - Vesicoureteral reflux
 - Urinary tract infection
 - Pain
- Instructions**

Attention! Do not use stone baskets if the stone fragment to be removed is too large for endoscopic removal.

- Insert the stone basket, closed, into the target organ using the uretero-renaloscopy.
- Release and unfold the basket by advancing the scissors handle to the tip of the instrument.
- Carefully manipulate the urinary stone or stone fragments into the opened basket.
- Then withdraw and hold the scissors handle, securing the stone.
- Pull out the instrument, along with the uretero-renaloscopy, and remove the stone from the basket. Repeat this process until all stone fragments have been removed.

If there is any resistance when advancing or withdrawing the instrument, investigate the reason for this and take suitable measures to address the issue. If the instrument gets stuck, use scissors to cut the tube close to the handle. Pull out the uretero-renaloscopy and re-insert next to the instrument. Reduce the stone to small pieces mechanically or using a laser, and remove the instrument.

- Possible complications and/or risks**
 - Very large kidney stones may be too large for removal using a stone basket and can get stuck in the kidney area or ureter.
 - With very large stones, it may not be possible to remove the stone from the basket in some circumstances.
 - Basket inverting
 - Breakage of the basket during forceful withdrawal despite resistance
 - If the basket is strongly bent directly at the tube-basket connector, the advancement and withdrawal of the basket may be impeded or impossible. In this event, the entire stone basket should be advanced or withdrawn a few millimeters. Do not advance or pull back the basket against strong resistance, otherwise there is a risk of damage to the product.

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- Warnings / Precautions**
 - This product is only sterile if the packaging is undamaged and unopened.
 - For single use only! Do not re-use, recycle, or re-sterilize. Re-using, recycling, or re-sterilizing can impair the structural integrity of the instrument and/or lead to malfunctioning of the instrument, which can in turn lead to illness, injury, or death of the patient.
 - Do not use products after the date of expiry has passed.
 - Do not wipe the product and set components with alcohol/disinfectant agents. This may damage the surface
 - All components must be carefully checked for compatibility and integrity before use.
 - The product is intended for a temporary service life of ≤ 60 minutes.
 - Do not rotate the open basket in the Ureter. A kink in the tube prevents the mechanical operation of the basket.
 - To prevent the basket from becoming stuck during withdrawal, it is recommend to expand the ureter before the endoscopy and stone removal.
 - The size of the stone should be determined by ultrasound or x-ray before the procedure to prevent it from getting stuck in the urinary tract. Do not use the stone basket if the stone is too large.
 - Push forward or remove the stone basket carefully to prevent damage to the mucous membrane or the urinary tract (trauma perforation).
 - With the simultaneous use of laser lithotripsy, great caution is required, as the nitinol basket can be very easily damaged by the laser. The laser should under no circumstances be fired directly into the basket.
- Interaction with other drugs**
 - The stone basket or its components could potentially be discolored when applying urinary examination methods that use dye.
- Transport and storage conditions**
 - The products may be transported and stored only in the packaging intended for this. There are no further specific requirements for transport. Products must be stored dry and protected from direct sunlight, in the temperature range of 5 - 30 degrees Celsius.
- Disposal**
 - After use, this product may pose a biological hazard. Handling and disposal must be carried out in accordance with recognized medical procedures and be completed pursuant to applicable legal regulations and guidelines of the respective country.

After use, this product may pose a biological hazard. Handling and disposal must be carried out in accordance with recognized medical procedures and be completed pursuant to applicable legal regulations and guidelines of the respective country.

The products may be transported and stored only in the packaging intended for this. There are no further specific requirements for transport. Products must be stored dry and protected from direct sunlight, in the temperature range of 5 - 30 degrees Celsius.

After use, this product may pose a biological hazard. Handling and disposal must be carried out in accordance with recognized medical procedures and be completed pursuant to applicable legal regulations and guidelines of the respective country.

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