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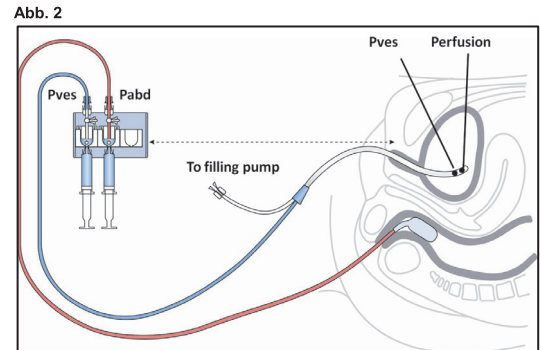
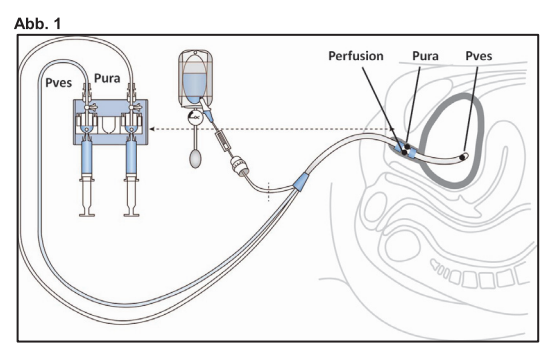
Table with 8 columns (DE, GB, FR, IT, ES, PT, CN) and 25 rows of symbols and their corresponding text in multiple languages.

GANL Nr. 30

Please note: This medical product may only be obtained and used by trained nursing and medical personnel.

- 1. Description
Double and triple lumen measurement catheter, double lumen rectal pressure catheter for urodynamics.
2. Contents and packaging
- Triple lumen measurement catheter with Tiemann or Nelaton tip, with or without measurement point marker
- Double lumen measurement catheter with Tiemann or Nelaton tip
- Double lumen measurement catheter with pigtail and stylet
- Double lumen rectal pressure catheter, silicone with balloon
- Double packaging and instructions for use

Sterilised with ethylene oxide.
3. Intended use
Measurement catheter for cystometry and urethral pressure profilometry (UPP) measurement.

- 4. Indications
Diagnostic indications:
- Vesical dysfunction, uninhibited detrusor contractions
- Urinary incontinence

5. Contraindications
Severe infections, Urosepsis.
Contraindications arise from restrictions on the corresponding basic applications

6. Side effects
Not known

7. Instructions
a. Connect the measurement catheter according to the color codes provided on it.
Triple lumen measurement catheter
- Transparent: Filling channel
- Blue: Measurement channel (Pves)
- Yellow: Measurement channel (Pura)

- Double lumen measurement catheter
- Transparent: Filling channel
- Blue: Measurement channel (Pves)
Double lumen rectal pressure catheter
- Female luer-lock for water filling till water level is seen in the middle of the transparent catheter.
- Insert catheter.
- 2 ml additional filling.
- filling the balloon with specified volumina



b. Connection to the measuring station
- Figure 1 for the triple lumen measurement catheter
- Figure 2 for the double lumen measurement catheter and rectal pressure catheter

8. Possible complications and/or risks
- Do not treat the products with disinfectant agents, since UROVISION has no information regarding possible material changes that this may cause.

- 9. Warnings / Precautions
- Product is only sterile when the packaging is undamaged or unopened. For single use only!
- All components must be carefully checked before use. Products that may be damaged must not be used.
- This product should be used only by physicians experienced in the use of such medical devices.
- Do not wipe the product with alcohol. This could cause damage to the surface
- Do not over-block the balloon! Only block to the maximum volume specified! (Details can be found on the catheter attachment and/or on the label)

10. Interaction with other drugs
Not known

11. Disposal
After use, this product may pose a biological hazard. Handling and disposal must be carried out in accordance with recognized medical procedures and be completed pursuant to applicable legal regulations and guidelines.

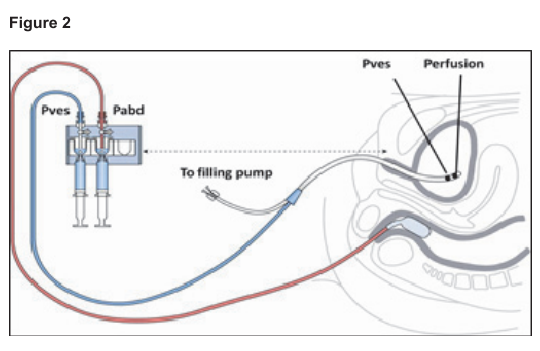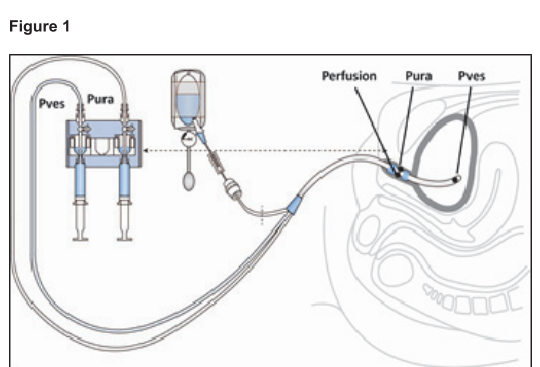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